# BLANCONE®

CLICK

Quick post hygiene treatment	GB	2
Trattamento rapido post igiene orale	IT	4
Quick-Behandlung nach der Mundhygiene	DE	6
Traitement rapide post-hygiène dentaire	FR	8
Tratamiento rápido post higiene bucal	ES	10
Tratamento rápido pós-higiene oral	PT	12
Snelle behandeling na de mondhygiëne	NL	14
Hurtig efterhygiejnebehandling	DA	16
Nopea valkaisu suuhygieniakäynnin yhteydessä	FI	18
Rask tannbleking etter tannpleie	NO	20
Snabb efterföljande hygienbehandling	SV	22
Szybki zabieg przeprowadzany po higienie jamy ustnej	PL	24
Rychlé ošetření ihned po dentální hygieně	CZ	26
Rýchle ošetrovanie zubov po ústnej hygieny	SK	28
Gyors poszt-higiénia-kezelés	HU	30
Tratament rapid post igienă orală	RO	32
Быстрое отбеливание после полости рта	RU	34
Ağız hijyeni sonrası hızlı tedavi	TR	36
معالجة سريعة بعد تنظيف الأسنان.	AR	38

[www.blancone.eu/pro](http://www.blancone.eu/pro)

## Quick post hygiene treatment

BlancOne® CLICK is a quick post dental hygiene whitening treatment. This treatment offers a very popular upgrade to the dental hygiene session at a very low cost and in just 10 minutes.

BlancOne® CLICK has been designed to promote a complete whitening treatment (Touch or Ultra) and as a promotional and marketing tool for the office.

### USE INSTRUCTIONS

#### A) Preparing the gel


1. Remove the caps of the two syringes (whitening gel and activating gel) and connect them together using the luer-lock connector.
2. Gently push the piston of one syringe to completely extrude the gel into the other syringe.
3. Now push the piston of the syringe with the gel, transferring it into the empty syringe.
4. Repeat the operation described in the previous step until the two gels (whitening and activator) are completely mixed, creating a single homogeneous gel.
5. Remove the connector from the syringe with the gel and attach the applicator tip.

#### B) Preparing the patient

1. The patient should be fitted with a BlancOne® gag with tongue depressor and then insert cylindrical tampons above and below the lips to isolate them from the area to be treated.
2. Remove the salivary film with a brush; if necessary, use a mix of water and pumice stone. DO NOT use polishing pastes or products containing fluoride. Rinse and dry.
3. Measure the patient's initial shade of white using the shade guide.

#### C) Whitening treatment

1. Gently push the piston of the syringe containing the whitening gel to apply the gel on the surface of all teeth to be bleached.
2. Cure the whitening gel with a blue light.  
Once activated, the gel will be swollen and full of oxygen (bubbles) as shown in the figure.

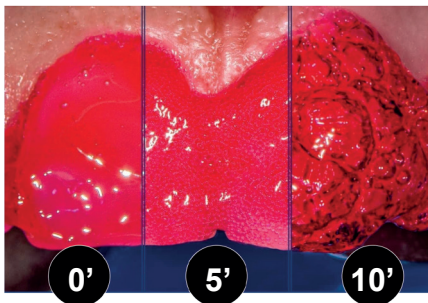3. Remove the excess whitening gel using a cylindrical cotton roll and suction away the gel residues, rinsing with the air/water syringe.

**WARNING:** The activated gel will be rather dense. If the suction spout becomes clogged, suction off with copious amounts of water.

**NOTE:**

the gingival edge might be slightly red in some points but this is a temporary effect that will disappear in about one hour.

#### D) End of treatment

1. Use the shade guide to measure the final shade of white of the patient's teeth.
2. Remove the gag and rinse the patient's mouth.

#### Photo activation of the gel

BlancOne® whitening gels require a suitable light source for the photochemical activation.

To guarantee the ideal whitening result, we recommend using BlancOne® ARCUS whitening lamp, which, being equipped with 10 powerful 5W led lights, guarantees the full activation of the gel simultaneously on both arches, including the sides, within the short time expected.

The head of BlancOne® ARCUS lamp should be positioned perpendicularly to the teeth at a distance of 5-10 cm from the mouth to guarantee a good ventilation.

Using other whitening lamps might reduce the effectiveness of the treatment, require a longer time or, in some cases, completely fail to activate the whitening gel.

#### RECOMMENDED ACTIVATION TIME

Equipment	Power	Type of activation	Time
BlancOne® ARCUS Lamp	250 mW/cm <sup>2</sup>	simultaneous on both arches	10 minutes

#### EUROPEAN UNION WARNINGS

- 5,6% of H<sub>2</sub>O<sub>2</sub> is released.
- Not to be used on anyone under 18 years of age.
- To be sold only to dental practitioners.
- To be used only by dental practitioners or under their direct supervision if an equivalent level of safety is ensured. Only then can it be made available to the patient, to complete the course of treatment.

#### PRECAUTIONS

- A dental examination is mandatory prior to bleaching.
- Bleaching will not lighten porcelain crowns and dental restorations.
- For at least 6 hours after the bleaching procedure do not use tobacco products and do not drink coffee or red wine.
- Not intended for use by pregnant women. Consult a doctor if you are breast feeding.
- Do not swallow the gel. Avoid contact with tissues other than teeth.
- Acidic foods/juices may cause sensitivity if consumed shortly after bleaching.
- Gum tissue irritation may result if used in individuals with periodontal disease.
- Keep teeth bleaching products away from children.
- Avoid contact with skin, mucous membranes and eyes. In case of contact with EYES: rinse eyes carefully with water for several minutes and request medical advice/attention.

ON SKIN/ MUCOUS MEMBRANES: wash with copious amounts of water. If irritation occurs request medical advice/ attention.

- If swallowed in small quantities the product is harmless, otherwise consult a doctor.
- Some users may experience sensitivity or gingival irritation. While these are mild and transient in most cases, sometimes treatment by a dentist may be required.

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